


Medtronic

Engineering the extraordinary

VersaOne Access System

Trocars

Sept - Oct
T&E Trainer

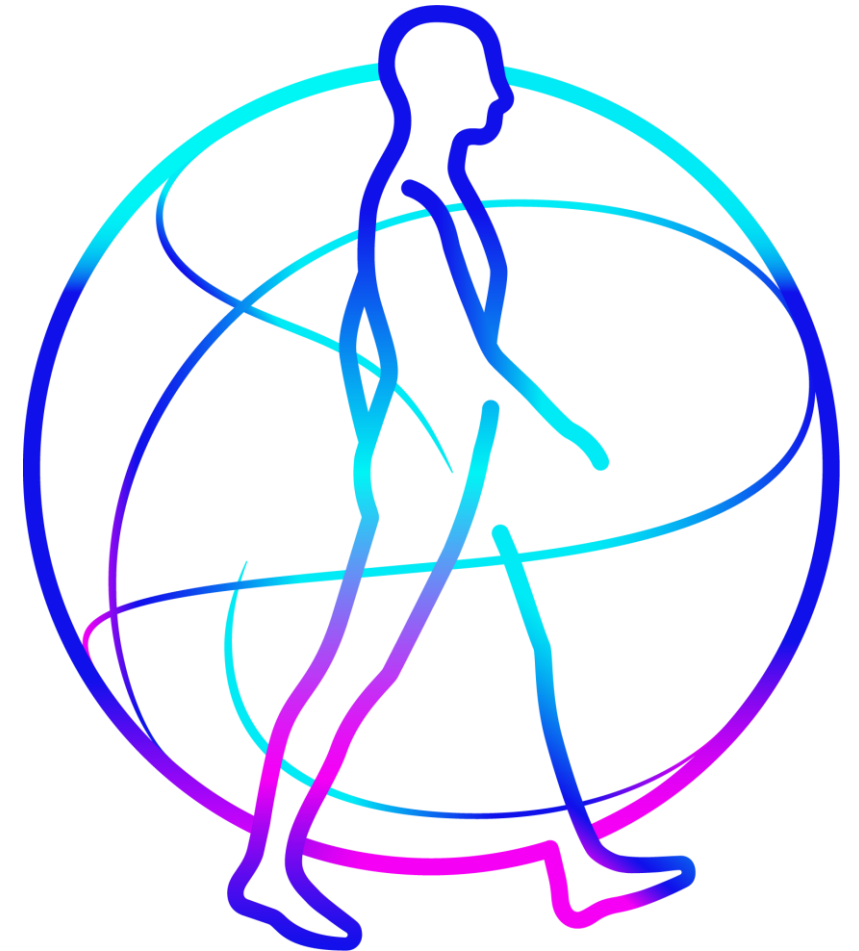


1 VersaOne™ ACCESS SYSTEM

LESSON

VersaOne™ Universal Cannula

VersaOne™ Obturators

VersaOne™ Trocars

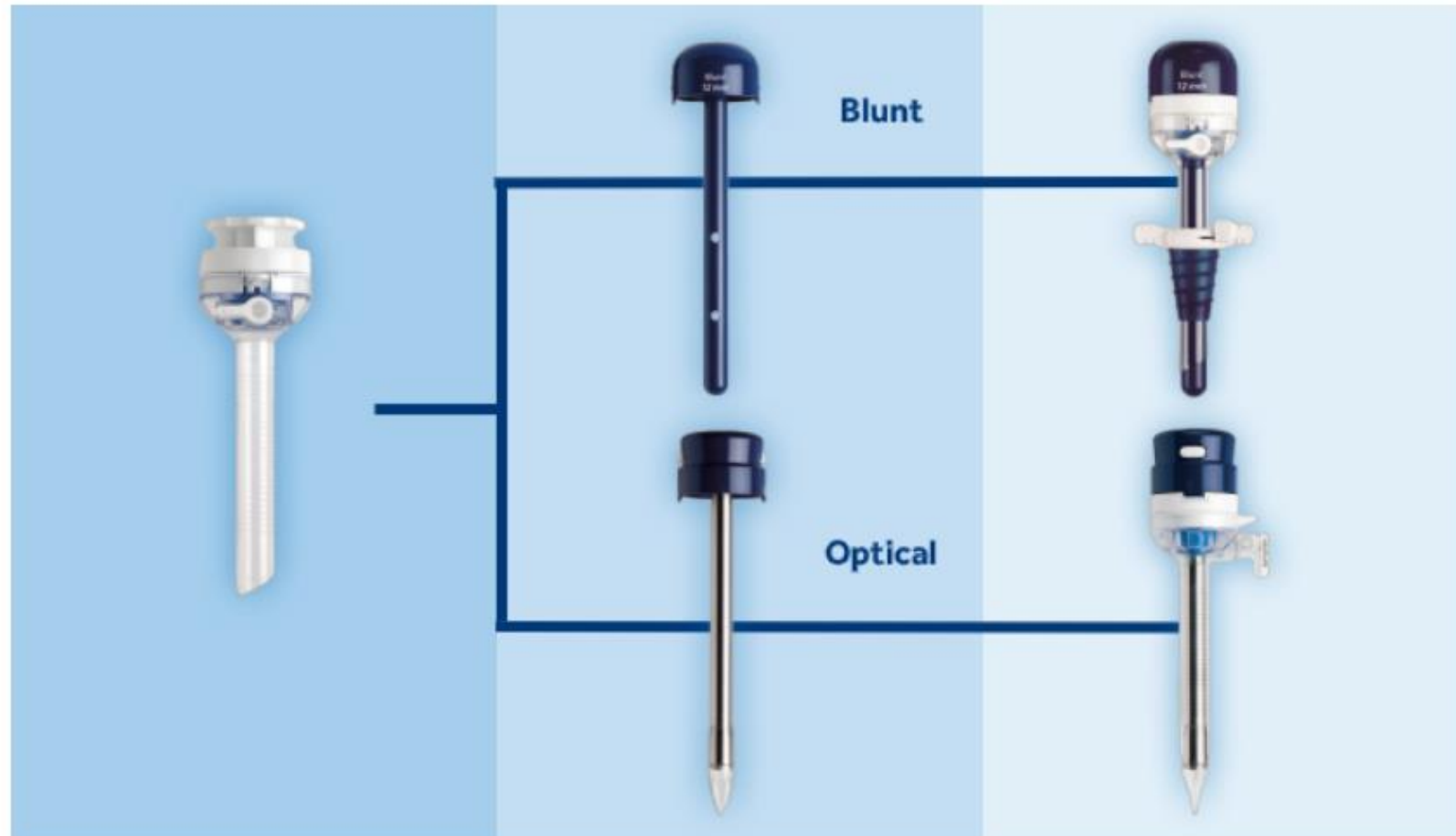

2 VersaOne™ UNIVERSAL CANNULA SPECS

LESSON


Three diameter sizes:



Three lengths:




Cannula type:

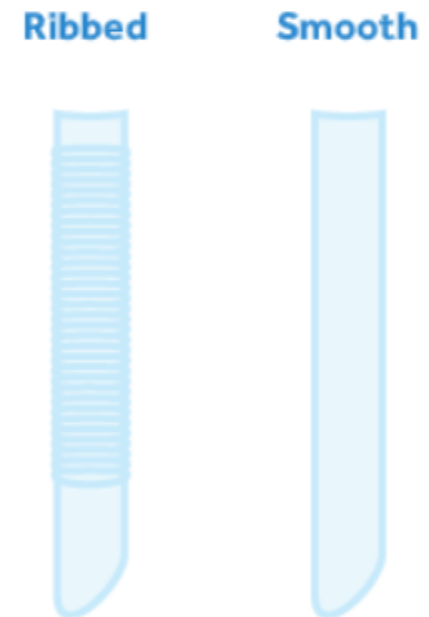


2 5 mm VersaOne™ UNIVERSAL CANNULA

LESSON FEATURES AND BENEFITS

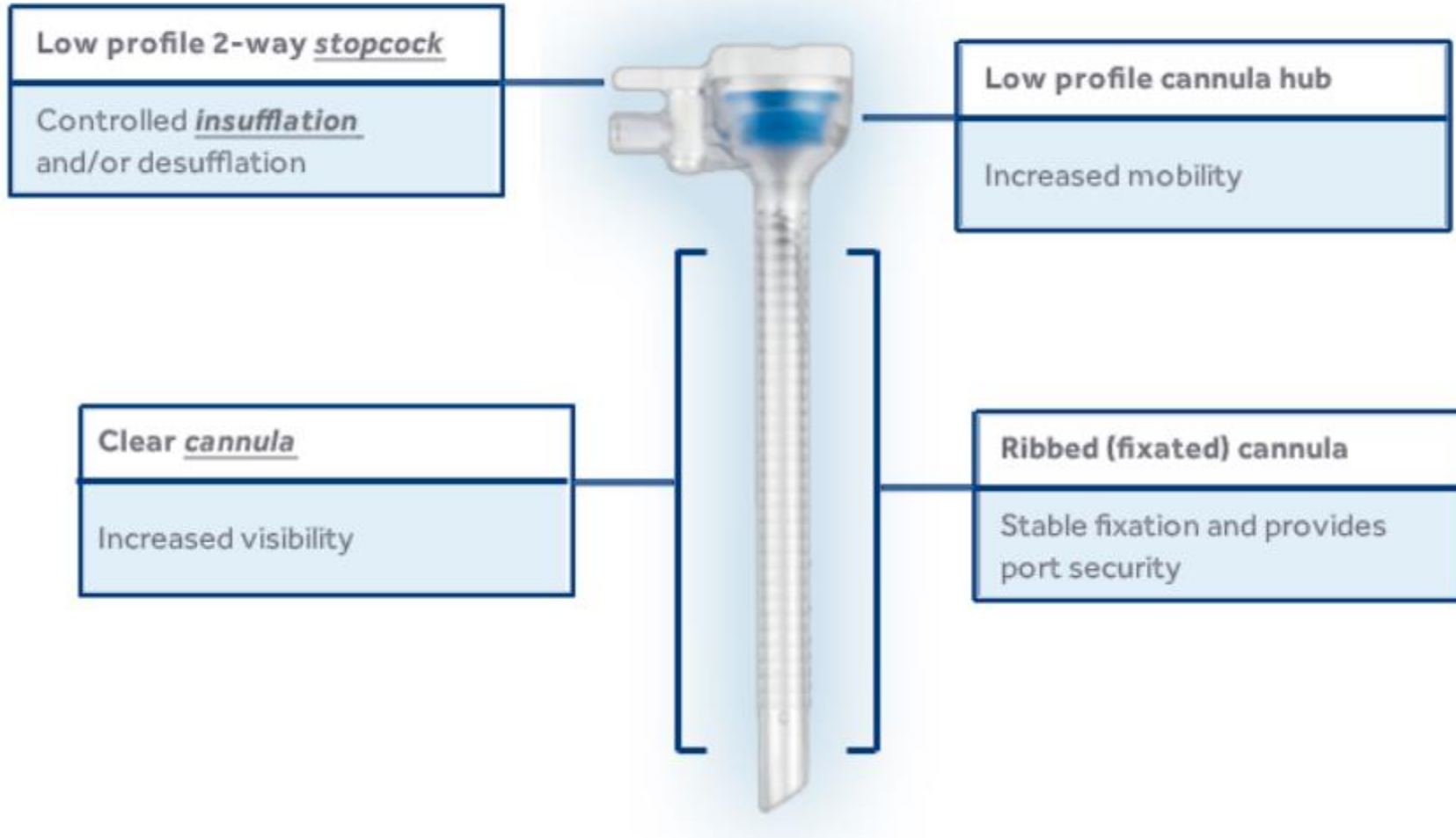

2 UNIVERSAL CANNULA

LESSON FEATURES AND BENEFITS

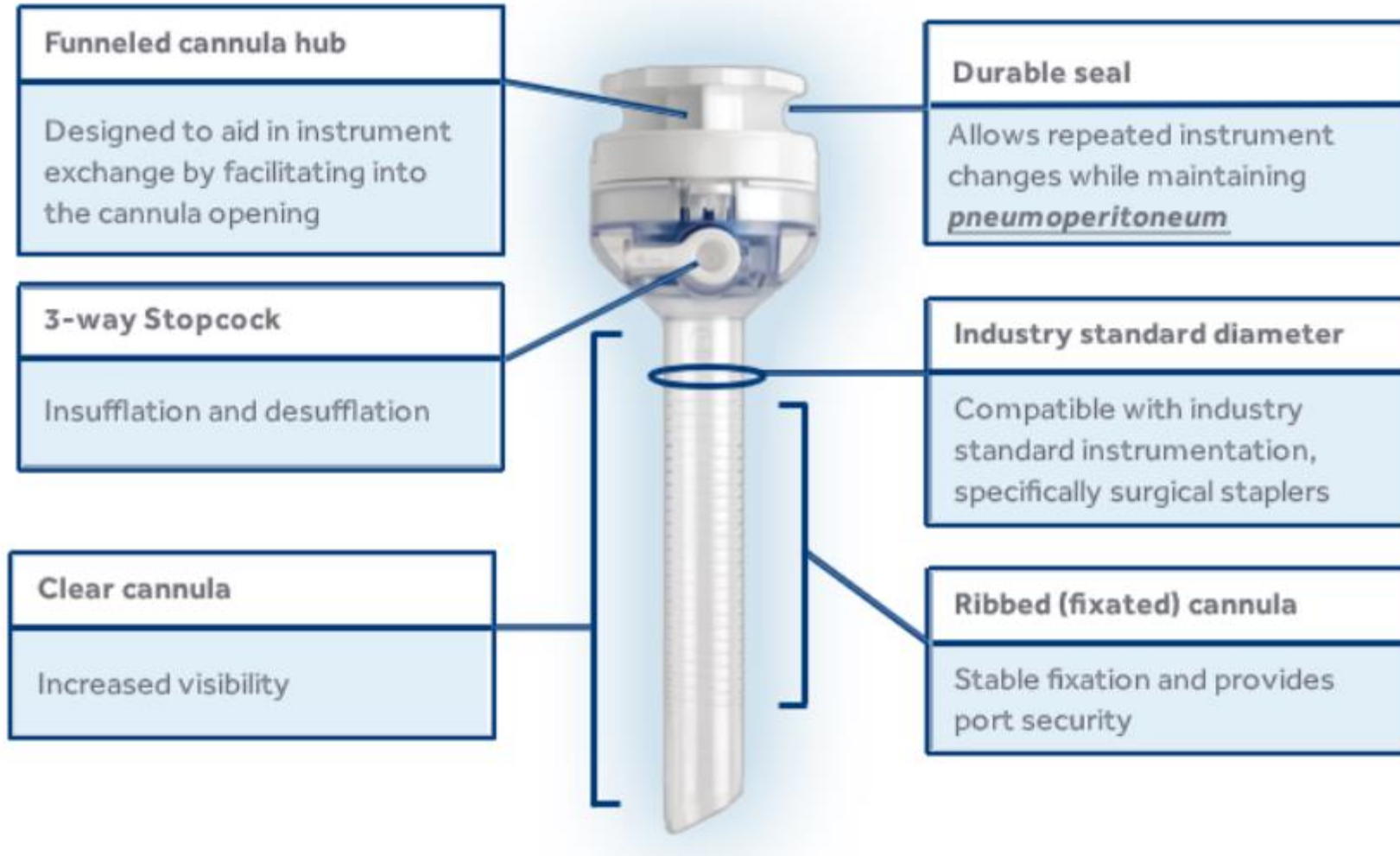

VersaOne Optical Trocar

4 VersaOne™ OPTICAL TROCARS

LESSON

VersaOne™ Universal Cannula



VersaOne™ Obturators



VersaOne™ Trocars




4 VersaOne™ OPTICAL TROCARS

LESSON DESCRIPTION AND FUNCTION


VersaOne™ Optical Trocars




Three lengths:

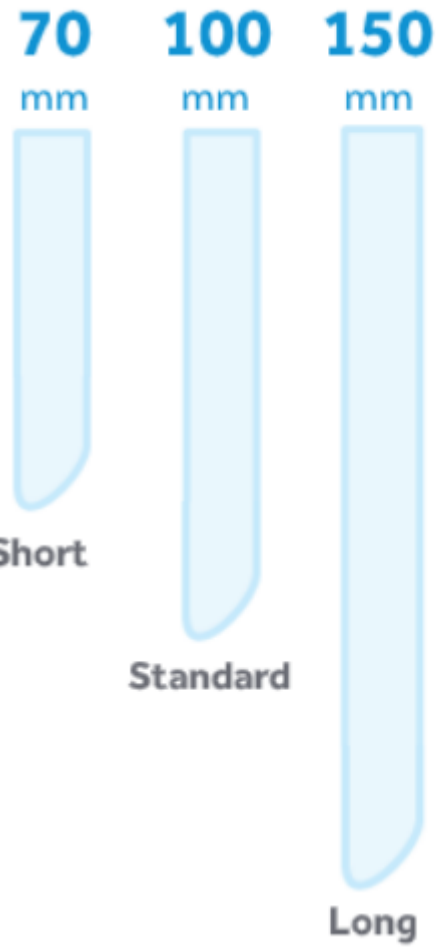


Cannula type:

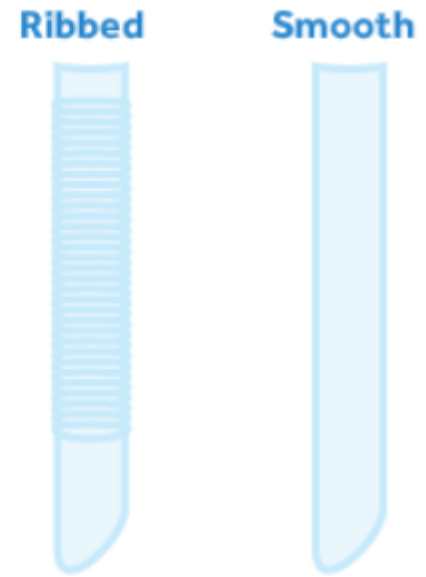

Optical Obturator and Housing

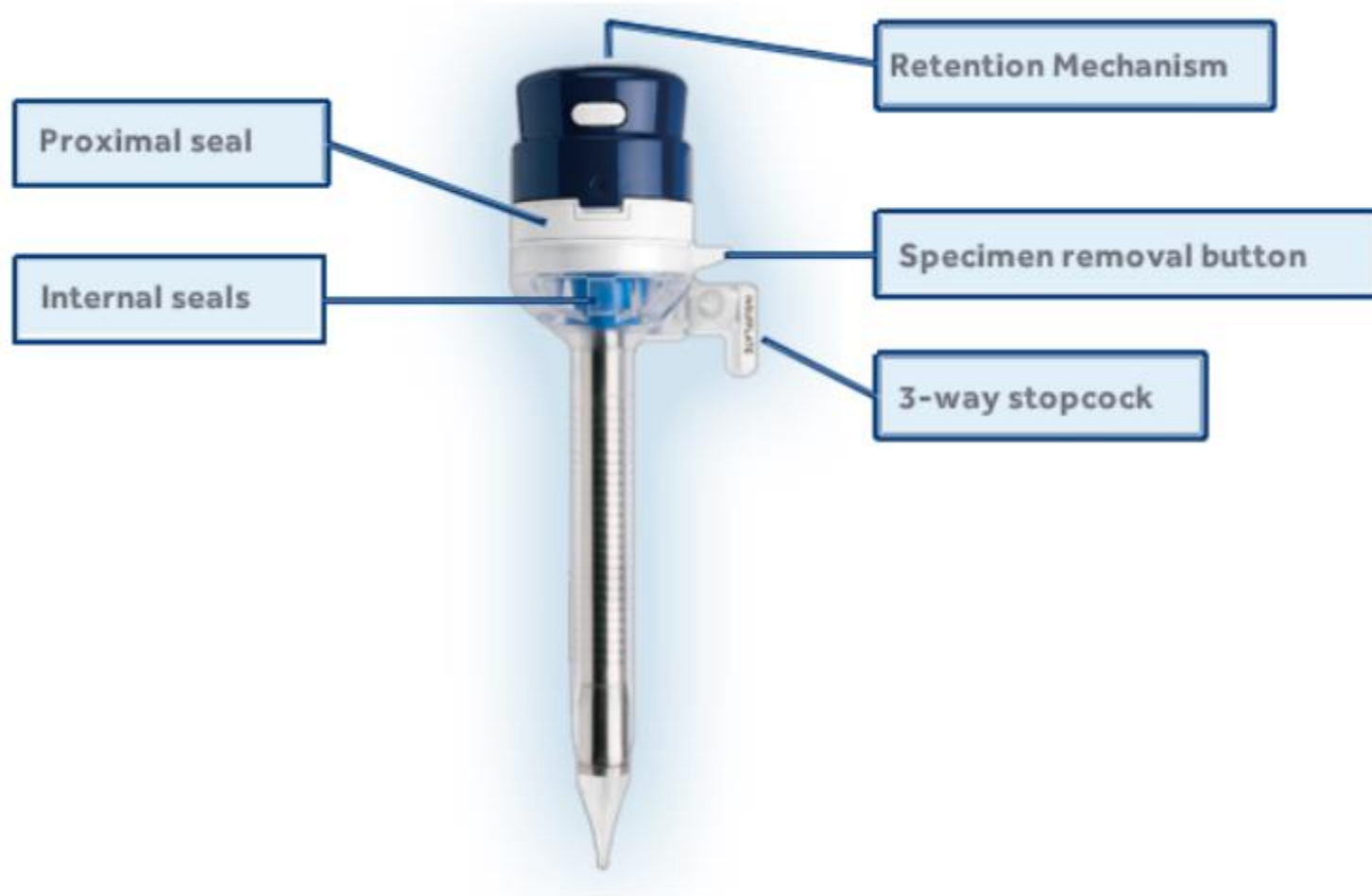


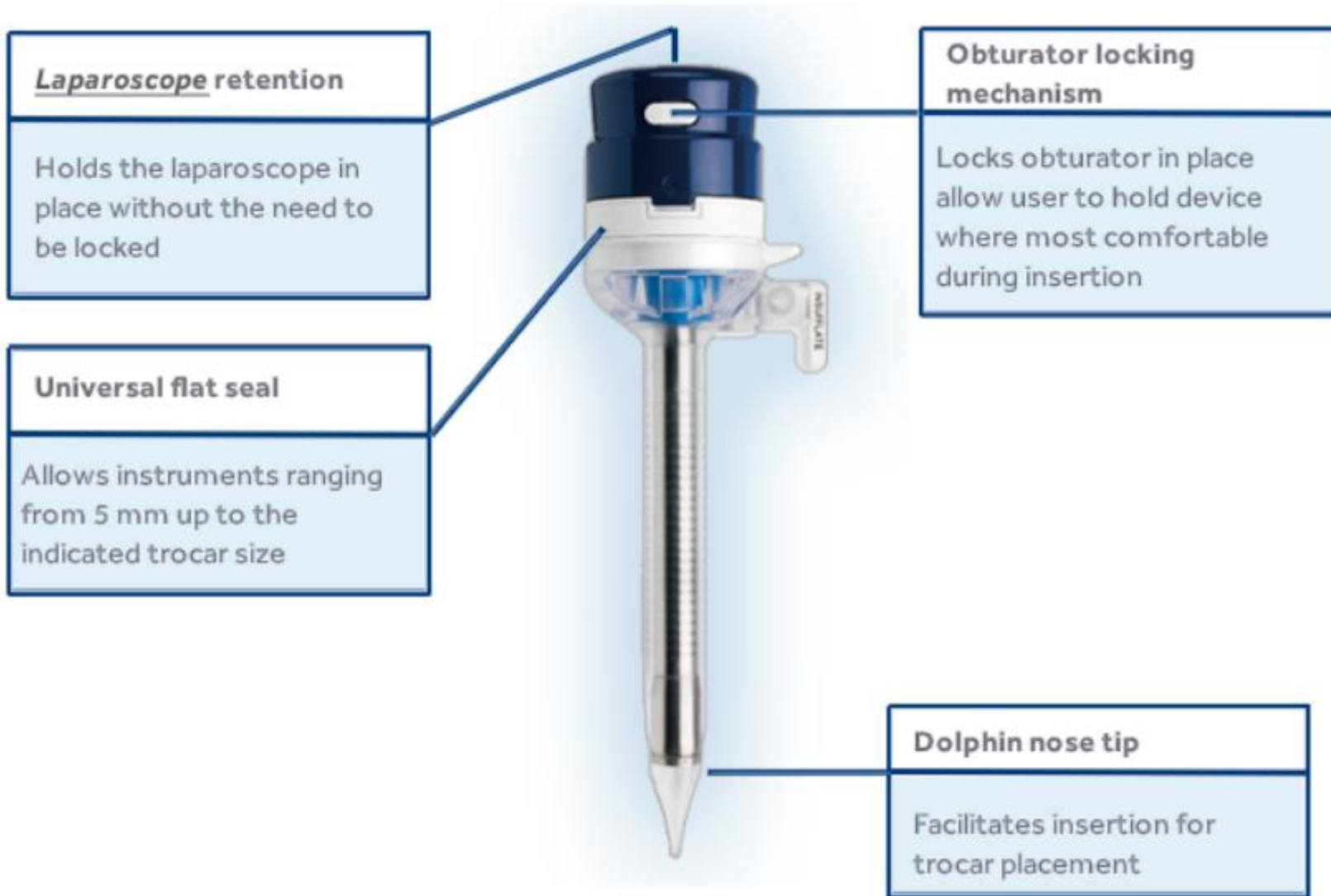

4 VersaOne™ OPTICAL TROCARS

LESSON INDICATIONS

The VersaOne™ Optical Trocars are intended for use in a variety of gynecologic, general, thoracic and urologic laparoscopic procedures to create and maintain a port of entry. The trocar may be used with or without visualization for primary and secondary insertions.

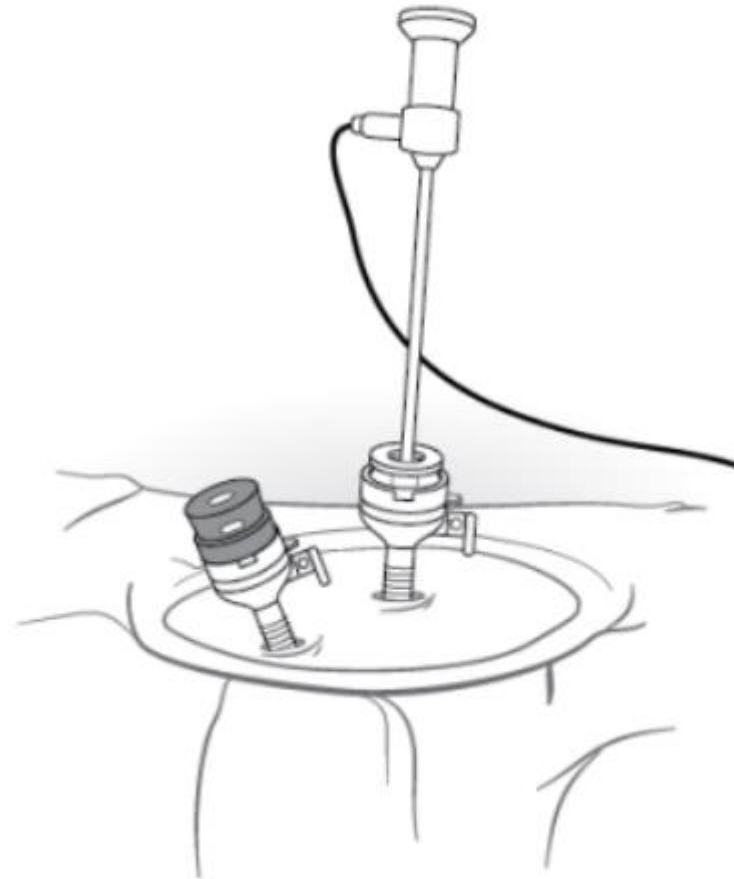

Primary Steps

1. Insert obturator
2. Connect laparoscope
3. Focus camera
4. Create pneumoperitoneum
5. Insert trocar into the peritoneal cavity
6. Remove obturator
7. Reinsert laparoscope/Instruments
8. Attach gas line
9. Remove cannula

VersaOne™ Optical Trocars



ORDER CODE	DESCRIPTION	BOX QTY
ONB5SHF	VersaOne™ Optical Trocar with Fixation Cannula, Short Length 5 mm	6
ONB5STF	VersaOne™ Optical Trocar with Fixation Cannula, Standard Length 5 mm	6
ONB5STF2C	VersaOne™ Optical Trocar with an Extra Standard Fixation Cannula, Standard Length 5 mm	6
ONB5LGF	VersaOne™ Optical Trocar with Fixation Cannula, Long Length 5 mm	6
ONB11STF	VersaOne™ Optical Trocar with Fixation Cannula, Standard Length 11 mm	6
ONB11LGF	VersaOne™ Optical Trocar with Fixation Cannula, Long Length 11 mm	6
ONB12SHF	VersaOne™ Optical Trocar with Fixation Cannula, Short Length 12 mm	6
ONB12STF	VersaOne™ Optical Trocar with Fixation Cannula, Standard Length 12 mm	6
ONB12STS	VersaOne™ Optical Trocar with Smooth Cannula, Standard Length 12 mm	6
ONB12LGF	VersaOne™ Optical Trocar with Fixation Cannula, Long Length 12 mm	6
ONB15STF	VersaOne™ Optical Trocar with Fixation Cannula, Standard Length 15 mm	6

VersaOne™ Universal Cannula



ORDER CODE	DESCRIPTION	BOX QTY
UNVCA5SHF	VersaOne™ Universal Cannula, Fixation, Short Length 5 mm	6
UNVCA5STF	VersaOne™ Universal Cannula, Fixation, Standard Length 5 mm	6
UNVCA5LGF	VersaOne™ Universal Cannula, Fixation, Long Length 5 mm	6
UNVCA11STF	VersaOne™ Universal Cannula, Fixation, Standard Length 11 mm	6
UNVCA12STF	VersaOne™ Universal Cannula, Fixation, Standard Length 12 mm	6
UNVCA12STS	VersaOne™ Universal Cannula, Smooth, Standard Length 12 mm	6
UNVCA12LGF	VersaOne™ Universal Cannula, Fixation, Long Length 12 mm	6

VersaOne™ Bladed Trocar

Single Use Trocar System with Universal Seal



ORDER CODE	DESCRIPTION	BOX QTY
B5SHF	VersaOne™ Bladed Trocar with Fixation Cannula, Short Length 5 mm	3
B5SHS	VersaOne™ Bladed Trocar with Smooth Cannula, Short Length 5 mm	3
B5STF	VersaOne™ Bladed Trocar with Fixation Cannula, Standard Length 5 mm	3
B5STS	VersaOne™ Bladed Trocar with Smooth Cannula, Standard Length 5 mm	3
B11STF	VersaOne™ Bladed Trocar with Fixation Cannula, Standard Length 11 mm	3
B11STS	VersaOne™ Bladed Trocar with Smooth Cannula, Standard Length 11 mm	3
B12STF	VersaOne™ Bladed Trocar with Fixation Cannula, Standard Length 12 mm	3
B12STS	VersaOne™ Bladed Trocar with Smooth Cannula, Standard Length 12 mm	3
B12LGF	VersaOne™ Bladed Trocar with Fixation Cannula, Long Length 12 mm	3
B12LGS	VersaOne™ Bladed Trocar with Smooth Cannula, Long Length 12 mm	3

VersaOne™ Bladeless Trocars

Port fixation and security

ORDER CODE	DESCRIPTION	BOX QTY
NONB5SHF	VersaOne™ Bladeless Trocar with Fixation Cannula, Short Length 5 mm	6
NONB5STF	VersaOne™ Bladeless Trocar with Fixation Cannula, Standard Length 5 mm	6
NONB5LGF	VersaOne™ Bladeless Trocar with Fixation Cannula, Long Length 5 mm	6
NONB8STF	VersaOne™ Bladeless Trocar with Fixation Cannula, Standard Length 8 mm	6
NONB11STF	VersaOne™ Bladeless Trocar with Fixation Cannula, Standard Length 11 mm	6
NONB12SHF	VersaOne™ Bladeless Trocar with Fixation Cannula, Short Length 12 mm	6
NONB12STF	VersaOne™ Bladeless Trocar with Fixation Cannula, Standard Length 12 mm	6
NONB12STS	VersaOne™ Bladeless Trocar with Smooth Cannula, Standard Length 12 mm	6
NONB12LGF	VersaOne™ Bladeless Trocar with Fixation Cannula, Long Length 12 mm	6



Thank you

Q&A